



Using Achromat™ Esthetic Fiber Posts:



1. Preparation of the canal shown with the appropriate Endodontic Drill to match the chosen post size.

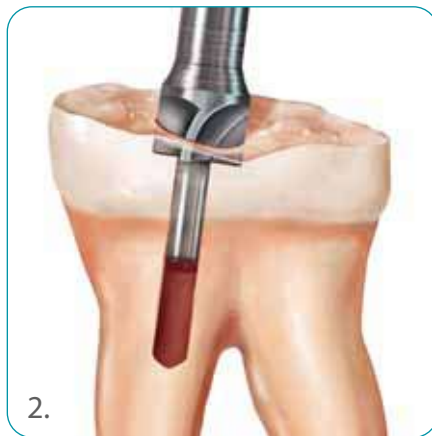

2. If using the Achromat-HP™, with the retentive "arrowhead" design, prepare the coronal surface with the appropriate root facer to match the size of the post used to achieve a countersunk and level seating of the post.

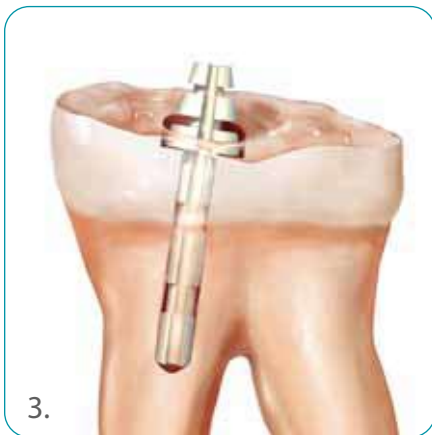

3. Try in the post. Confirm that the base of the retentive "arrowhead" design lies flat on the pulpal floor. Proceed to bonding the post into the canal by following the manufacturer's directions for the chosen bonding product.



AXIS DENTAL CORPORATION
800 Sandy Lake Road, Coppell, Texas 75019
Customer Service: 800.355.5063
Product Specialist: 888.654.2947
972.536.6000 • FAX: 972.257.3647
www.axisdental.com

AXIS Dental Products are exclusively marketed by AXIS Dental Corporation and are available through Authorized Dental Dealers only. AXIS, ACHROMAT, and ACHROMAT-HP are trademarks of AXIS Dental Corporation. © 2003 AXIS Dental Corporation. Printed in USA.

AXIS Dental also offers a full line of NTI Gates Drills and NTI Peeso Reamers.

L0405-ACHRO